

Product Installation Guide Sheet

Herbst Appliance Caps and Bumpers



Comfort Cap

Comfort Bumper



Wet first with water and roll forward to back until completely on.



Spring Appliance Cap Products



2 - 4 mm Caps to be used on 29 mm or larger Rods



Posterior Spring Cap

Anterior Spring Cap



Using the scaler tool, pierce the top of 2 Comfort Cap 4 mm, then place the 2 - 4 mm Caps facing each other onto the 29 mm or larger Rod.



Visit www.comfortsolutions.ca for product ordering or contact information.

HTH CAP




Wet with water first for lubrication. Slide cap up rod and over end till securely on.


FLIP LOCK CAP



SPI- WRAP TUBING (MULTI-USE) Insert end on and spin on to appliance

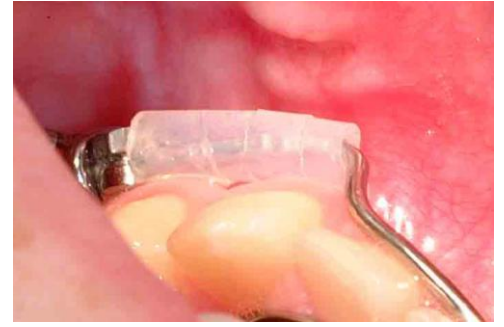

CUSPID CAP

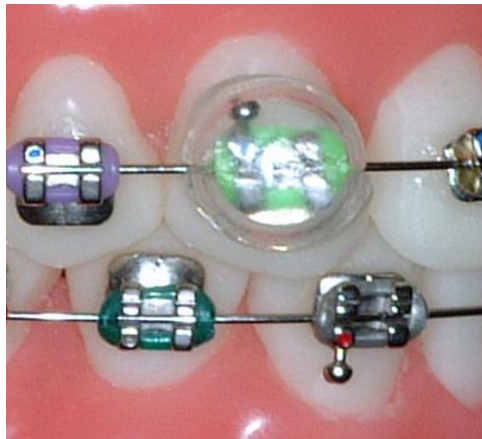

Usage when no wire inserted or when Bracket Cap Clips cannot hook on wire. Note slot for wire.

BRACKET CAP



Clips over wire for stability of Occlusion of upper arch. Either for Incisors or Bicuspid areas.

MARA CAP

Insert lower CAP First

