


Product Installation Guide Sheet



Xbow™ shown with Forsus™ Fatigue Resistant Device from 3M Unitek

New! Forsus™ EZ Module with Cap

Phase 1.

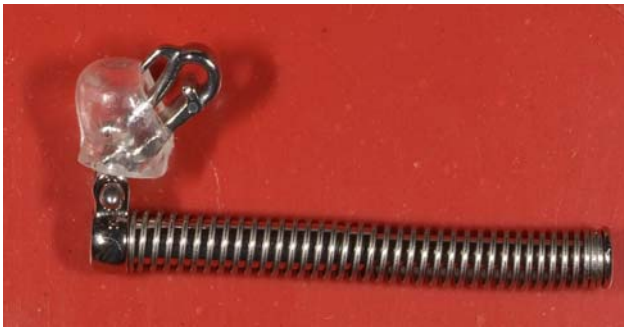

Phase 2.



1. If Forsus Spring is already attached, slide Cap over Spring with Separating Plier.



2. Slide down spring to proper position



For buccal mucosa sores place two anterior caps, one on the spring and one on the pushrod.

Final Position



Herbst Appliance Caps and Bumpers



Comfort Cap

Comfort Bumper



Wet first with water and roll forward to back until completely on.



HTH CAP

FLIP LOCK CAP



Wet with water first for lubrication. Slide cap down rod and over end till securely on.

SPI- WRAP TUBING (MULTI-USE)
Insert one end and spin on to appliance.



BRACKET CAP



Clips over wire for stability of Occlusion of upper arch. Either for Incisors or Bicuspid areas.


CUSPID CAP



Usage when no wire inserted or when Bracket Cap Clips cannot hook on wire. Note slot for wire.

MARA CAP

Insert lower CAP First

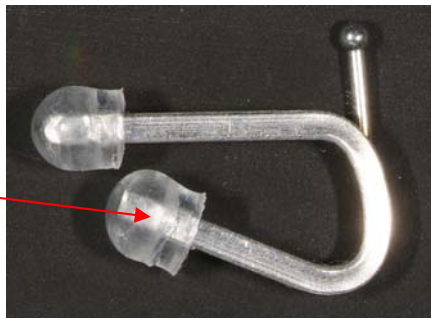

TAD CAP

